## **MERCATOR** MEDICAL



# dermagel® coated

The instruction below should be used in conjunction with detailed information on the packaging.

#### Short description of the product

Natural rubber latex examination and protective gloves, powder-free, non-sterile for disposable use

#### Full description of the product

Raw material · natural rubber latex

External surface · textured Cuff · beaded Colour · creamy

: ambidextrous, fitting to the right and left hand Shape Size range : XS (5-6), S (6-7), M (7-8), L (8-9), XL (9-10)

AOI · 1 0

Quantity in packaging : 100 pcs. by weight

Shelf life : 3 years (from the date of manufacturing)

#### Storage instructions

It is recommended to store the gloves in dry place, in the temperature of 5-35°C and to protect them against direct sunlight and fluorescent light. Recommended relative humidity in the room where the gloves are stored is

Keep the gloves in a distance of not less than 1m from heating devices, sources of fire and ozone.

Do not keep in direct vicinity of solvents, oils, fuels and lubricants.

#### Food contact

Gloves are marked with food contact symbol  $\overline{X}$  and comply with the requirements of Regulation (EU) No 10/2011, European Regulation (EC) No 1935/2004 and with Regulation (EC) No 2023/2006 on Good Manufacturing Practice. Gloves are suitable for handling any type of food, except for acidic foods, and have been tested for Overall Migration Test acc. EN 1186:

Extraction conditions	Analysis results	Test Result
(tested for 2 h in 40°C)	[mg/dm <sup>2</sup> ]	(limit < 10 mg/dm <sup>2</sup> )
3% acetic acid	>10.0	No Pass
10% ethanol	3.2	Pass
Olive Oil	8.6	Pass

#### MDR classification & compliance

Gloves are classified as class I according to Annex VIII of the Regulation (EU) 2017/745 and comply to standards:

EN 455-1:2000, EN 455-2:2015, EN 455-3:2015, EN 455-4:2009, EN ISO 15223-1:2016, EN 1041:2008+A1:2013.

#### PPER classification & compliance

Gloves are category III Personal Protective Equipment as per Annex I of the Regulation 2016/425 and comply to standards:

EN 420:2003+A1:2009, EN ISO 374-1:2016 (Type B), EN 374-2:2014, FN 16523-1:2015, FN 374-4:2013, FN ISO 374-5:2016.

Declaration of Conformity can be found under below web address:

https://mercatormedical.eu/products/gloves/examination-and-protectivegloves/dermagel-coated

**Notified Body 2777** responsible for EU Type Examination (Module B) and Module C2 On-going Conformity: Satra Technology Europe Ltd Bracetown Business Park, Clonee Dublin 15, Dublin, Ireland



#### Intended use

These are non-sterile examination and protective gloves for single use, intended for use in medical field to: protect patient and user from crosscontamination, conducting medical examinations, diagnostic and therapeutic procedures and for handling medical contaminated material. Gloves are classified as Medical Devices Class I and as a Personal Protective Equipment Category III. Their design and labelling corresponds to the requirements of the European Regulation 2017/745 on Medical Device and the European Regulation 2016/425 on Personal Protective Equipment. Gloves should be used solely according to their intended application.

#### Precautions and indications for use

Dry hands before putting the gloves on. Before usage, inspect the gloves for any defect or imperfections. Use at least 1 pair of gloves for one patient and one procedure, these are disposable gloves. Do not let chemical substances get under the gloves through the cuff. If a chemical substance reaches the skin, wash it away immediately with plenty of water with soap. If the gloves get punctured, torn or broken during their use, take them off and put on the new ones. Avoid using gloves dirty in the inside as they may cause irritation leading to skin inflammation or more serious damages. The gloves should not be used in contact with open fire and to protect against any sharp tools. The gloves are not intended for welding, electric shock protection, ionizing radiation or from the effect of hot or cold objects.

This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals. Degradation results indicate the change in puncture resistance of the gloves after exposure to challenge chemical. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in case where glove is equal to or over 400 mm - where the cuff is tested also) and relates only to the chemical tested. It can be different if the chemical is used in a mixture. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen.

It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on the temperature, abrasion and degradation.

When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves.

#### Components / hazardous components

Some gloves may contain components known to be a possible cause of allergy for person allergic to them, who may develop contact irritation and/or allergic reaction. In case of an allergic reaction, seek medical assistance immediately.

#### Disposal

Used gloves can be contaminated with contagious or other hazardous substances. They should be disposed of in accordance with local regulation. Gloves should be buried or burned under controlled conditions.

#### Manufacturer

MERCATOR MEDICAL S.A. ul. H. Modrzeiewskiei 30 31-327 Cracow, Poland www.mercatormedical.eu

## **MERCATOR** MEDICAL



Permeation performance levels as per EN ISO 374-1:2016									
• Level 1 > 10 min • Level 2 > 30 min • Level 3 > 60 min • Level 4 > 120 min • Level 5 > 240 min • Level 6 > 480 min									
Test results acc. to EN 16523-1:2015		EN 374-4:2013	Test results acc. to EN 16523-1:2015		EN 374-4:2013				
Chemical	Level	Degradation [%]	Chemical	Level	Degradation [%]				
1.5% Methanol	6	6.1	50% Sulphuric acid	6	-22.1				
10% Acetic acid	1	-1.4	5% Ethidium bromide	6	-24.7				
50% Benzalkonium chloride*	2	-10.7	3% Hydrogen Peroxide	6	-2.6				
4% Chlorhexidine digluconate**	6	-13.1	30% Hydrogen Peroxide (P)	2	4.3				
10% Phosphoric acid	6	-18.4	37% Formaldehyde (T)	5	-23.6				
40% Sodium hydroxide (K)	4	-59.2	5% Glutaraldehyde	6	-8.9				
12% Sodium hypochlorite	6	-25.4	0.1% Phenol	6	-32.7				

\*minimum detectable permeation rate: 5 µg/cm²/min

\*\*minimum detectable permeation rate: 7 μg/cm²/min

EN 374-4:2013 Degradation levels indicate the change in puncture resistance of the gloves after exposure to the challenge chemical.

Elitari i interes problemento i interes manere una suan be in parietara resolutario en una biorres artes exposare to una suan en						
Test acc. To EN 374-2:2014 – Level 2 (ISO 2859)		Test acc. To EN ISO 374-5:2016				
Performance level	AQL	Protection against bacteria & fungi	Pass			
Level 3	< 0.65	Protection against viruses	Pass			
Level 2	<1.5					
Level 1	< 4.0					

#### Symbols used on the packaging



Medical device



Personal Protective Equipment



Powdered gloves



Do not re-use / gloves are intended for single use



Keep away from moisture, store in a dry place



Powder free gloves



Non-sterile gloves



Keep away from solar and fluorescent light



Presence of polymer coating on the inner surface of the glove



Lot / batch number



Temperature limitation / gloves store in temperature



Presence of cosmetic coating on the inner surface of the glove



Catalogue number



Keep away from ozone



Presence of external texture on the glove



EU Authorised Representative, symbol should be accompanied by name and address of Authorised Representative



Date of manufacture



Gloves made from nitrile



Expiry date



Manufacturer, symbol should be accompanied by name and address of Manufacturer







Food contact symbol



Gloves made from vinyl





Marking of gloves protecting against bacteria and fungi



(article is suitable for food contact for details check the instruction for use)



Raw material natural rubber latex





Marking of gloves protecting against viruses, bacteria and fungi



Package made from paper, qualify for recycling



50 gloves by weight







Marking of type A chemical resistant gloves. Six tested chemicals shall be identified by their code letter under pictogram



Package is treated as municipal waste



100 gloves by weight





Marking of type B chemical resistant gloves. Three tested chemicals shall be identified by their code letter under pictogram



Consult instructions for use



200 gloves by weight



UVWXYZ



their code letter under pictogram

Marking of type C chemical

resistant gloves. One tested chemicals shall be identified by



Additional information on inner side of package



Do not use, if package is damaged



Indicates compliance with the requirements of Russian market



Indicates compliance with the requirements of Ukrainian market

## **MERCATOR** MEDICAL



### ■ HOW TO PUT THE GLOVES ON? ■













#### ■ HOW TO TAKE THE GLOVES OFF? ■











